



How to start the **Digital Anatomy Creator Trial**

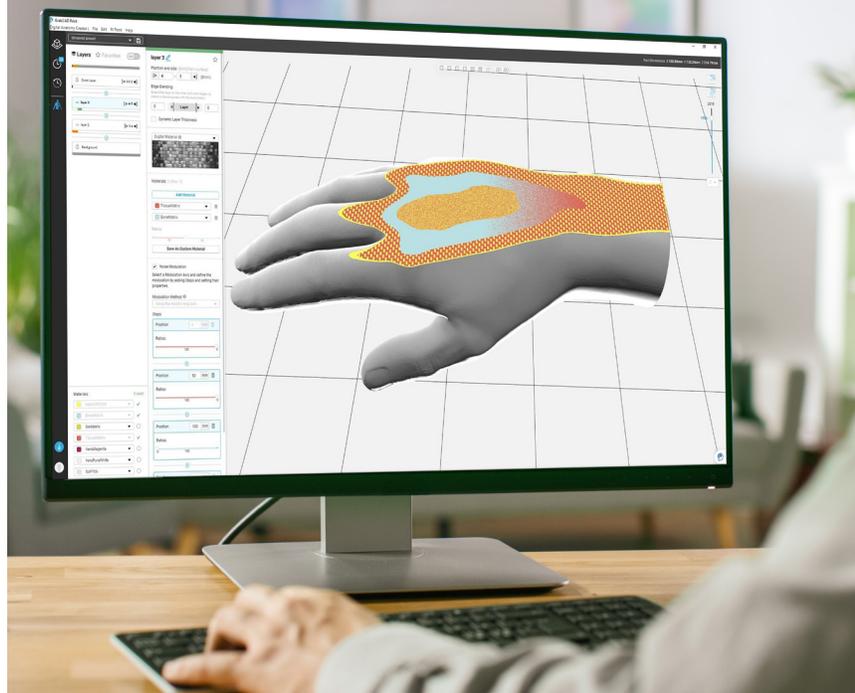

Do you own a Digital Anatomy™ printer
or are you a GrabCAD user?


YES I am already a GrabCAD or Digital Anatomy printer user

1. Launch GrabCAD Print and select a Template Printer
“Stratasys J750 Digital Anatomy”

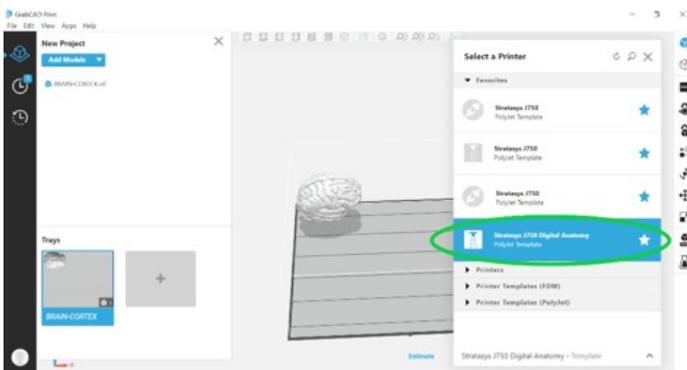

2. Load an STL file to the tray


3. Right click on the model and select “send to D.A.C”

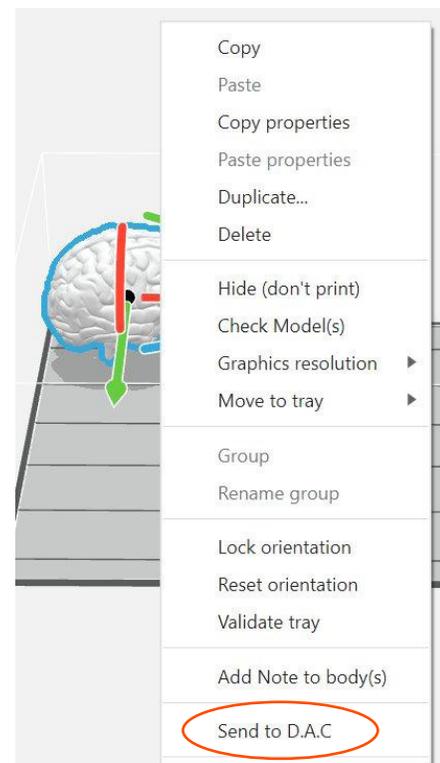


4. Use the DEMO mode to explore the
software features

NO I am not a GrabCAD or Digital Anatomy printer user

1. Signup for GrabCAD at <https://grabcad.com/print> and download GrabCAD Print.

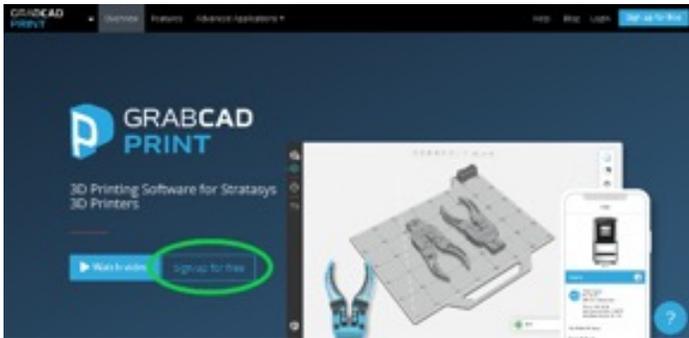


2. Launch GrabCAD Print and select a Template Printer “Stratasys J750 Digital Anatomy”

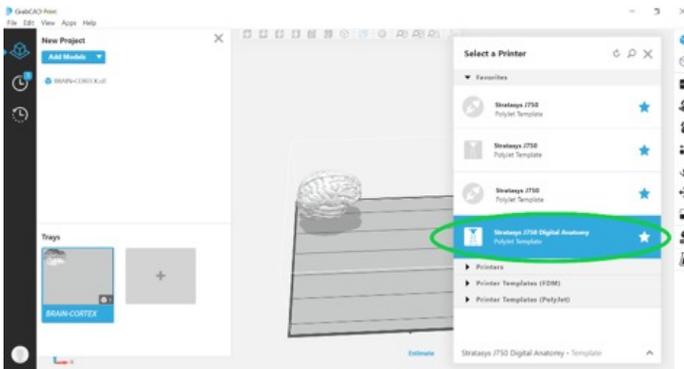


3. Load an STL file to the tray

4. Right click on the model and select “send to D.A.C”

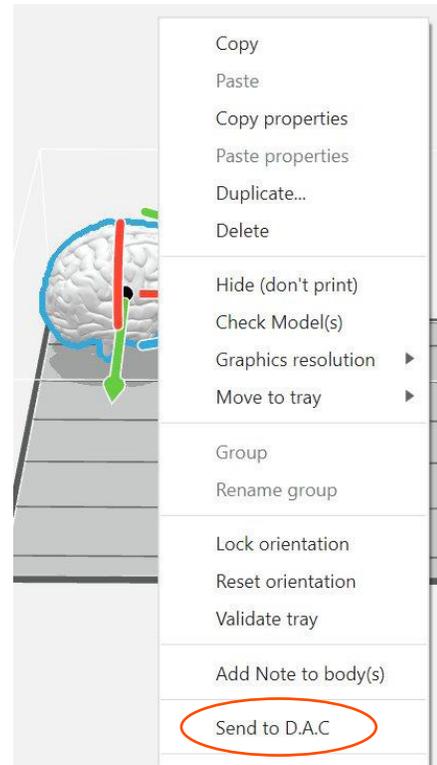

5. Use the DEMO mode to explore the software features

Note: The free trial (Demo Mode) provides full software capabilities apart for printing or saving a completed model design.
A Digital Anatomy Creator software license is required to save, share or print a model.

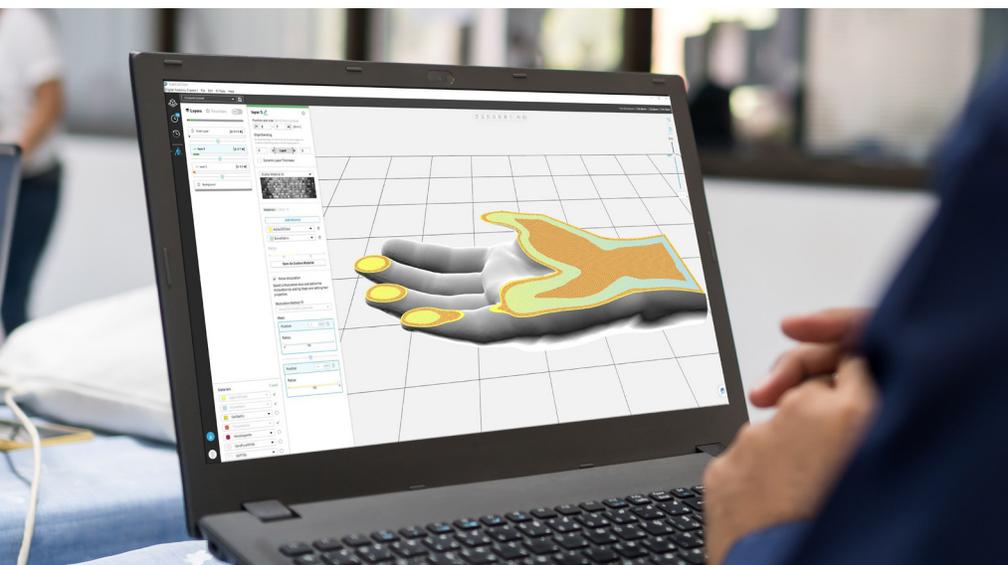

“

The Digital Anatomy Creator offers control to a mind-blowing level.”



Seth Friedman, PhD
Manager of Innovation Imaging
and Simulation Modeling,
Seattle Children's Hospital, Seattle, USA